


User manual

Tech Lift 500

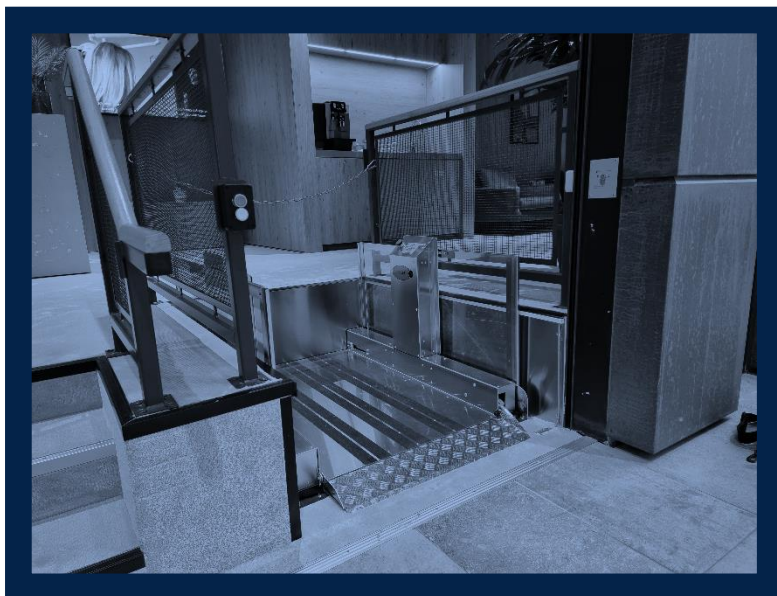


Tech Line Sweden AB

www.techline.nu

User manual for Tech Lift 500

Control panel

The lifting platform is controlled with a control panel. The lifting platform is made to be easy to use even without prior knowledge.

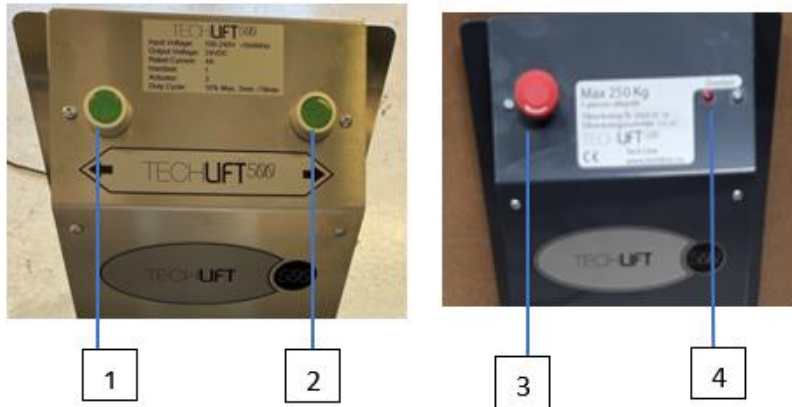

1. Down
2. Up
3. Emergency stop
4. Overload signal

Press and hold button 2 to raise the lift, the ramp folds up to prevent run-off.

Press and hold button 1 to lower the elevator.

In the event of an emergency, press the emergency stop, button 3. To release the emergency stop, turn the button clockwise.

If the lift is overloaded, the lamp lights up at 4.

This is the same when it run on UPS (battery backup)

Lifetime and service

Lifetime is based on proper use, cleaning and maintenance, as well as annual inspection and maintenance by qualified personnel or service personnel from your distributor.

At the end of the period, the Tech Lift 500 must be evaluated by qualified personnel to determine future use. Spare parts lists and drawings can be requested from the manufacturer or distributor.

Cleaning

Tech Lift 500 is cleaned with a sponge or brush and soapy water. Tech Lift 500 can be cleaned or rinsed under normal water pressure but must not be washed with high pressure washing. Never use acids, bases or solvents when cleaning.

What maintenance is required by the owner?

Normal cleaning.

That it is kept clean around the platform, so that the operation is not disturbed by strange subjects.

Technical specifications

Outside Dimensions (L x W)	1429 x 1050 mm
Inside dimensions (L x W)	1250 x 800 mm
Lift height	30 to 500 mm
Weight	51 kg
Maximum load	250 kg
Lifting speed	max 30 mm / sec
Electrical connection	230V
Power Consumption in Standby	0.3 W
Power Consumption in Operation	200 VA / 24V
Fuse	10A
Sound level	52 dB (A)

Sealing Class Lifting Platform Components IP 65

Is manufactured in accordance to:

- 2006/42/EG av 17 May 2006 – Machinery Directive
- 2004/108/EF av 15 December 2004 – EMC – Directive
- 2004/EF av 12 December 2006 – Low voltage directive

And in accordance with the relevant requirements of:

- EN-81-41:2010 Safety rules for the construction and installation of lifts. Special lifts for transport and persons and goods - Part 41: Vertical platform lifts intended for people with reduced mobility.

For service or questions about maintenance contact your distributor.